



## Care instructions

We have built the Easi-Scan remote display to be rugged and splash resistant, however, it does contain sensitive electronics. With a little care and maintenance you will get many years of excellent imaging.

The Easi-Scan Remote Display will be damaged if you:

- Drop it
- Hose it down
- Wipe the screen with your hand

For a long and happy life:

- Clean immediately after use — before any debris dries.
- Grit/mud should first be removed by rinsing gently — do not rub the screen.
- Dry excess water off the housing with clean soft cloth — do not rub the screen.
- Only wipe the screen with a clean cloth.
- Secure in protective carry case or neoprene sleeve once clean and dry.

## Specifications

<b>Display size:</b>	8.2" or 21 cm diagonal
<b>Luminance:</b>	800 cd/m <sup>2</sup>
<b>Contrast ratio:</b>	1000:1
<b>Battery life:</b>	5 hours
<b>Power source:</b>	Easi-Scan battery
<b>Environmental proofing:</b>	IP65
<b>Temperature range:</b>	-20 °C to +50 °C or -4 °F to +122 °F
<b>Weight (with Easi-Scan battery):</b>	1.65 kg / 3.64 lb
<b>To be used with:</b>	Easi-Scan and Duo-Scan

The Easi-Scan Remote Display has been designed to operate as a component of BCF ultrasound scanners and as such the combined system conforms with the requirements of European Directive 89/336/EEC relating to electromagnetic compatibility (EMC) and also 2002/95/EC relating to Reduction of Hazardous Substances (RoHS). If you require any further information regarding these directives please contact us directly.

Note: All BCF Technology products are to be used for animal applications only.

**UK +44 (0)1506 460 023**

**USA +1 (507) 529 8200**

**IRE +353 (0)42 932 0070**

**Australasia +61 (0)3 9894 8980**

For a list of BCF product distributors visit [www.bcftechnology.com](http://www.bcftechnology.com)